



EZE BLOODLESS CASTRATOR™

INSTRUCTIONS



Freephone NZ 0800 80 90 98 E: sales@technipharm.co.nz W: www.technipharm.co.nz
Freephone AU 1- 800 124 034 E: sales@technipharm.com.au W: www.technipharm.com.au

EZE BLOODLESS CASTRATOR™ - instructions

USING THE EZE BLOODLESS CASTRATOR™ WITH BAND AND CLIPS

We recommend the use of a well designed and safe Cattle Handler for any castrating work. If you do not have one visit our website www.technipharm.co.nz, www.technipharm.com.au or free phone 0800 80 90 98 or 1-800 124 034

APPLYING THE BAND

1. Grip the scrotum above the two testicles and gently "milk the sack" to separate the fatty tissue so that the band can be applied only onto the blood arteries and sperm tubes.
2. Failure to do this may mean that we do not get enough tension on the cord to cause the testicles to die & drop off.
3. Place the stretched rubber band as close to the testicles and as far below the belly as possible. When applying the ring, make sure that both testicles are below the band.

TIGHTENING THE BAND

4. Gripping the scrotum with one hand, pull the applicator away from the animal, causing the band to come away from the spreader bars on the unit.
5. The band is then looped only around the testicles and is attached to the shaft hook
6. Hold onto the side hand and pull the "T" handle as tight as you can.
7. Pump the "trigger" on the pistol grip until hook on the shaft is within 2 or 3 inches of the handle, or until sufficiently tight. Tightening the ring this tight, means that the blood flow to the scrotum will stop & that the scrotum will die & drop off.
8. You can then clamp band with the clip and cut the band

CAUTION! Make sure that band is removed from spreader bars before tightening. If the band is not removed from the spreader bars or if the ring is set too loose then there may be some restriction of blood but not enough to cause the testicles to drop off.

AFTER APPLYING THE BAND

9. After cutting the rubber ring, inspect the clip to make sure it is holding the rubber band correctly, if you are not happy place a second ring above the first one
10. The tighter the rubber ring is on the scrotum the better the results will be. If swelling occurs below the rubber band it is not tight enough. (Reband immediately). If the testicles do swell then bring the animal back to the yards and apply a second band
11. Where there is blood still reaching the scrotum, the scrotal sack may only partially separate from the body increasing the risk of fly strike and/or blood poisoning
12. After castration has been performed on the larger animals including rams. Drop time for scrotum to fall off is between 20 to 40 days. Some customers cut the scrotum off just below the clip in about 7 to 14 days.
13. If the scrotal sack has not dropped off naturally by day 40, it may pay to bring animals back to yard to check the status of the testicles,

BEFORE TRANSPORTING ANIMALS

14. 5 days prior to shipping any animals castrated with the Eze castrator, we recommend that a visually inspection of the animals takes place.

Ensure:

- That scrotal sack has fully detached from the animal,
- That there is no open wound where scrotal sack was attached to the body
- That there are no infections where scrotal sack was attached to the body

A TETANUS TOXOID INJECTION IS STRONGLY RECOMMENDED. Applying PINETAR to the scrotum will help protect from flies and infection.

The EZE™ also works well in the castration of Rams and Goats.

MAINTENANCE NOTES:

After some use the pullback rod may start to slip when tightening the band. Loosen the lock nut on the 1/4 inch screw, above and in front of the handle turn in one or two turns, and tighten.


WARRANTY

This product is warranted to one year from date of purchase for mechanical and material workmanship. The manufacture's obligation is limited to repairing this product during the warranty period.

Provided the product is sent prepaid to Technipharm International Ltd. This warranty does not cover the normal wear. Technipharm International Ltd its agents or assigns assume no liability for animals lost from the castration process.



EZE BLOODLESS CASTRATOR™ - instructions

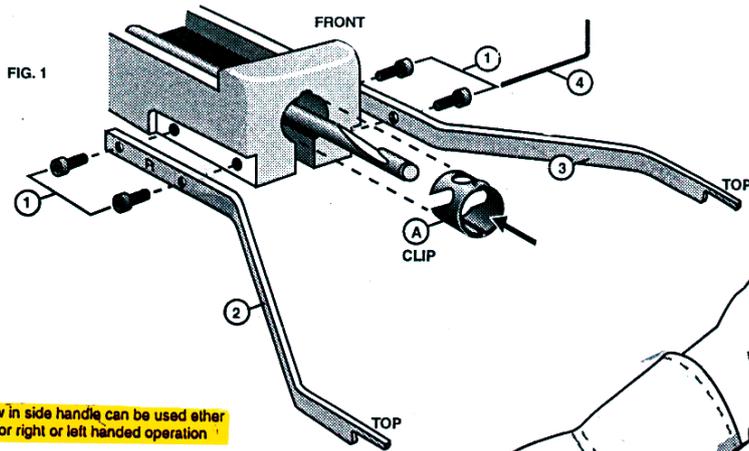

Screw in side handle can be used either side for right or left handed operation

INSTRUCTIONS:

1 PLACE CLIP (A) IN HOLE AT THE FRONT OF THE BLOODLESS CASTRATOR AS SHOWN WITH SIDE HOLE AT TOP AS SHOWN IN FIG.1




2 FOR BEST RESULTS HOLD EZE UNDER YOUR LEFT ARM FOR RIGHT HANDED PEOPLE. REST STAINLESS STEEL HANDLE ON YOUR FOREARM HOLDING RUBBER RING WITH YOUR LEFT THUMB. PULL RUBBER RING ACROSS TO THE OTHER TIP OF THE SPREADER BAR.

(STAINLESS STEEL SIDE GRIP HANDLE CAN BE REMOVED AND SCREWED IN TO RIGHT SIDE OF EZE FOR A LEFT HANDED PERSON.)

3 STRETCH RUBBER RING OVER TIP OF RIGHT SPREADER BAR


4 PULL RUBBER RING FROM BOTTOM OF LOOP OVER TOP OF SHAFT HOOK

5 SQUEEZE TRIGGER UNTIL RUBBER RING IS THROUGH THE CLIP.

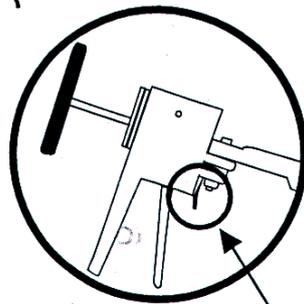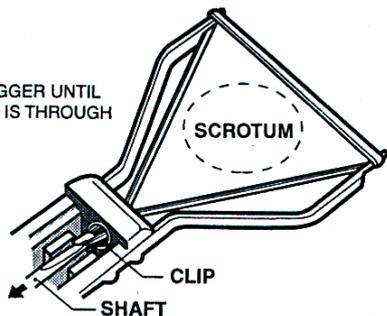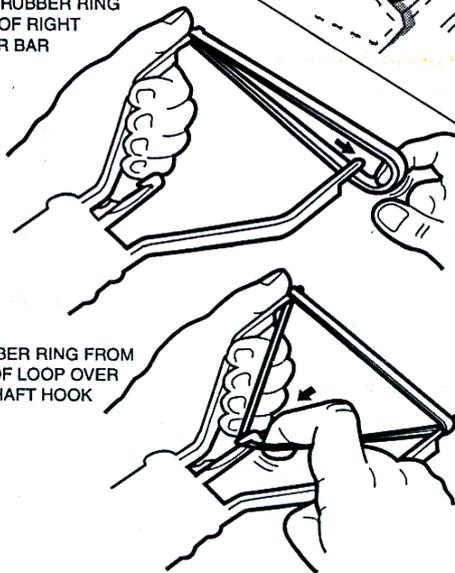

NOTE:
USE RELEASE TRIGGER ON EZE TO RELEASE "T" HANDLE FOR RESETTING.


LOOK ON BACK FOR TIPS AND WARRANTY

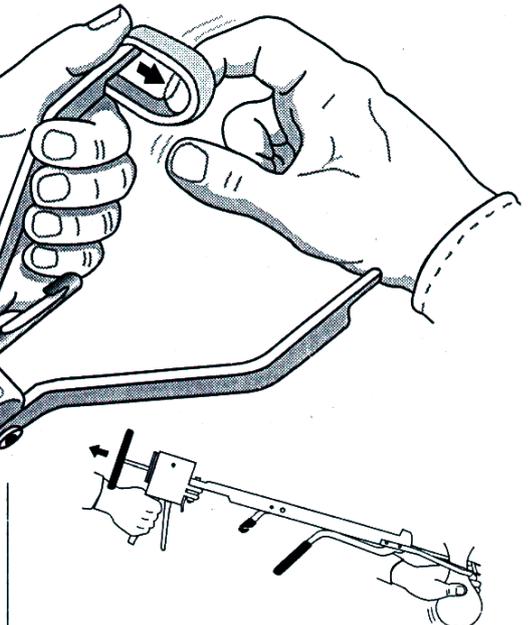

6 HOLD SCROTUM FORWARD AND PULL BACK ON EZE UNTIL RUBBER RING COMES OFF THE SPREADER BARS.



7 HOLD SIDE HANDLE AND PULL "T" HANDLE AT REAR OF EZE AS TIGHT AS YOU CAN.

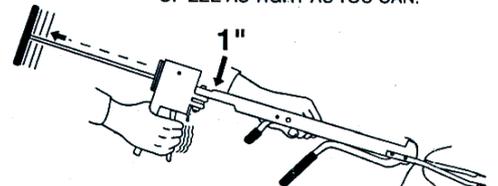

8 CONTINUE HOLDING SIDE HANDLE AND WITH YOUR OTHER HAND FINISH TIGHTENING UNTIL HOOK ON SHAFT IS WITH IN 1" OF HANDLE.



9 RELEASE SIDE HANDLE AND PUSH FRONT LEVER FORWARD TO CRIMP CLIP.



10 PUSH SMALL LEVER FORWARD TO CUT RUBBER RING.

EZE BLOODLESS CASTRATOR™ - instructions




1. Flush.



2. Hold band with thumb on top & index finger on end of spreader bar to keep from slipping off.



3. Continue holding while stretching with other hand.

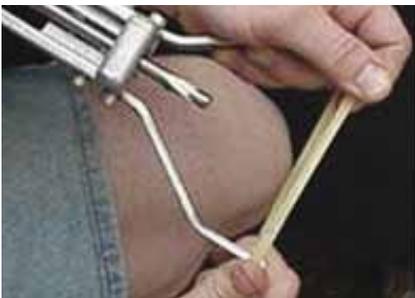

4. Stretch to other spreader bar and attach.





5. When stretching band, try to stretch the upper portion of the band as tight as possible, leaving the lower portion with as much slack as possible. Since this will be the portion that will be stretched the most.



6. Hold band with both thumbs on top and both index fingers in the end of spreader bars.

EZE BLOODLESS CASTRATOR™ - instructions

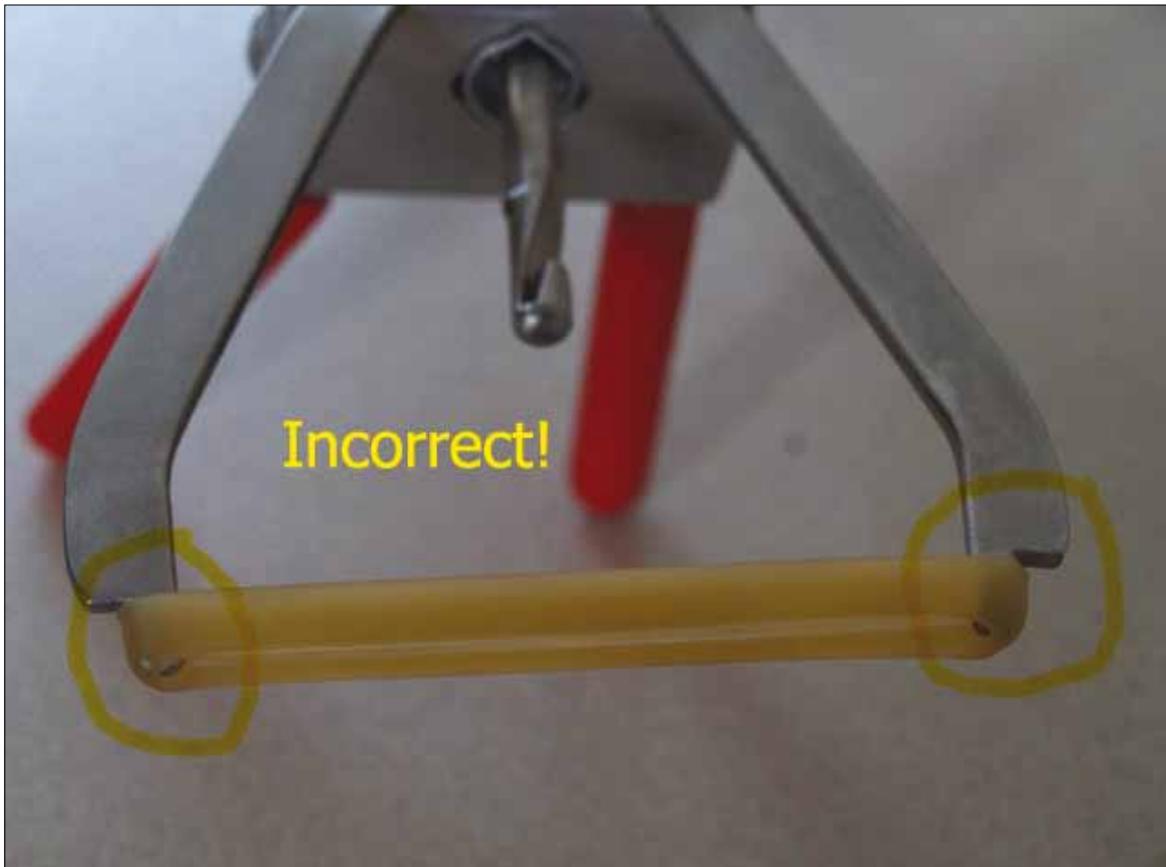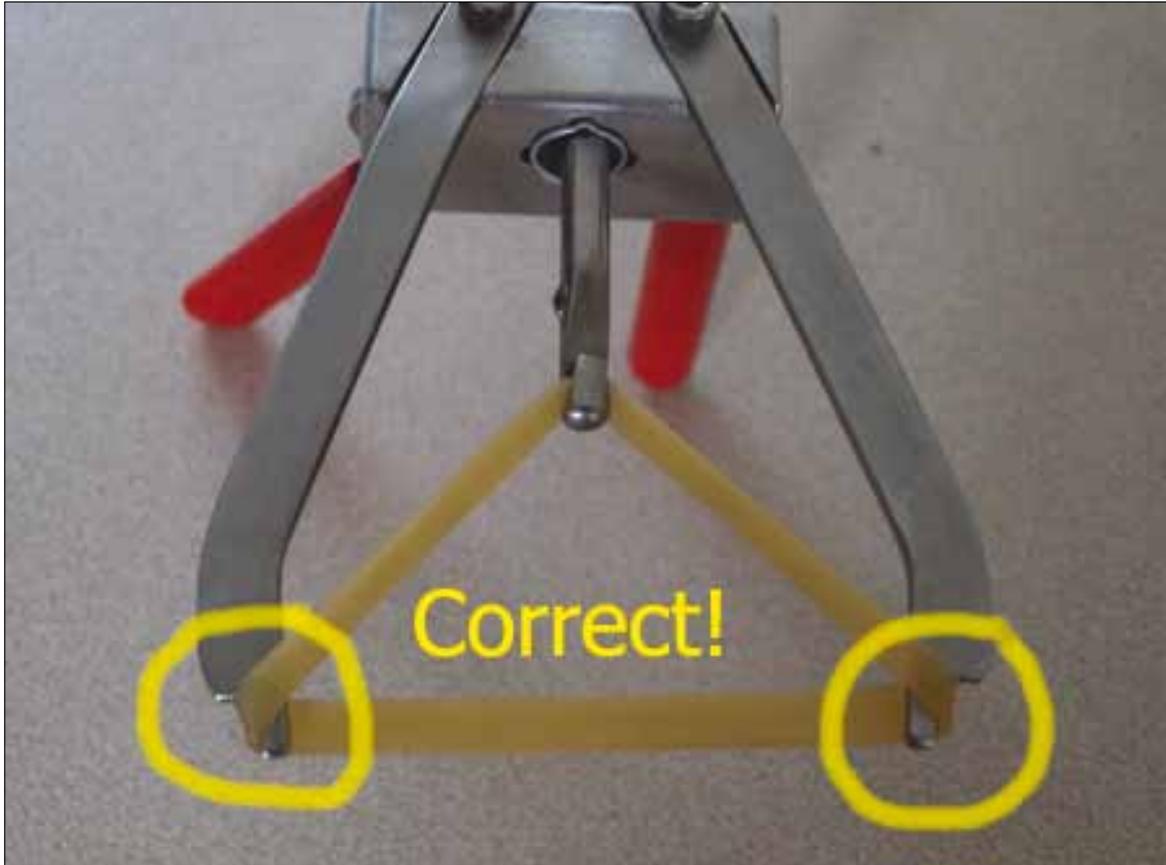



HEAD OFFICE
92/2 Hawthornden Drive
RD 4, ROTORUA
NEW ZEALAND

CORRESPONDENCE
P.O. Box 959
ROTORUA
NEW ZEALAND

SALES

NZ ☎ **0800 80 90 98**
📠 **(+64) 07 345 9643**
sales@technipharm.co.nz
www.technipharm.co.nz

AU ☎ **1-800 124 034**
sales@technipharm.com.au
www.technipharm.com.au

ADMIN. QUERIES/DELIVERY

☎ **(+64) 07 345 9642**
📠 **(+64) 07 345 9643**
admin@technipharm.co.nz

Qty	Castrators + Rings Order Form	Unit price	Total
	EZE BLOODLESS CASTRATOR™ includes 25 rings/clips FREE	\$895 +GST Ins/Freight	
	TIREX TRIBAND™ includes 100 rings FREE	\$325 +GST Ins/Freight	
	TRI-BAND GREEN™ includes 50 rings FREE	\$149 +GST Ins/Freight	
	50 pack rings and clips for the EZE BLOODLESS CASTRATOR™ 	\$189 +GST Ins/Freight	
	100 pack rings and clips for the EZE BLOODLESS CASTRATOR™ 	\$345 +GST Ins/Freight	
	50 pack rings for the Triband Green™ (or Triband Black™) 	\$26 +GST Ins/Freight	
	50 pack rings for the Tirez Triband™ 	\$26 +GST Ins/Freight	
	50 pack rings for the Elastrator HD™ 	\$26 +GST Ins/Freight	

Date:/...../..... Note: rings should be stored in a cool dark place.

Customer name:

Invoice to be made out to:

Address:

.....

.....

.....

Delivery Address (if different from above):

..... Ph:.....

email: Fax:.....

Preferred date of delivery(Date) Preferred method of delivery(Method)

PAYMENT: **Paid in Full \$**..... (Amount) Method:.....

(please tick) **DEPOSIT Paid \$**..... **Final amount of \$**..... will be paid on/...../..... Method:.....

Nothing Paid today. Payment of \$..... will be made on...../...../..... Method:.....

Net Total	\$
Credit card surcharge 3%	\$
Insurance 1%	\$
*Freight charge	\$
Sub Total	\$
GST	\$
Total Due	\$

OFFICE USE

DELIVERY: **Goods sent on:**...../...../.....(Date) By:.....(Method)

(please tick) **Still to be delivered:**.....

Other relevant details :

.....

.....

All orders will be subject to final approval by the Financial Controller.

This order complies with all our trading policy conditions (see reverse) and by placing the order I accept these.

*Note that most items will incur freight charges for which we'll invoice you at COST ONLY at a later date when cost is known.

This copy may be used as an official GST/TAX invoice. **GST # 92 957 055**

CONFIDENTIAL